

Emesis Bags

Easy emesis containment




Unlike emesis basins, emesis bags reduce risk of spills and help control odor. As the market leader, Medline offers a large selection of emesis bags and dispensers through every major GPO to help you provide a better, more sanitary experience for both patients and caregivers.

- Easy to use: simply twist closed, hook into notch and dispose
- Eliminates caregiver and patient contact with emesis
- Prevents spills and costly linen changes
- Convenient dispensers available—can be placed in any location
- Manufactured without natural rubber latex




Easy to install and dispense

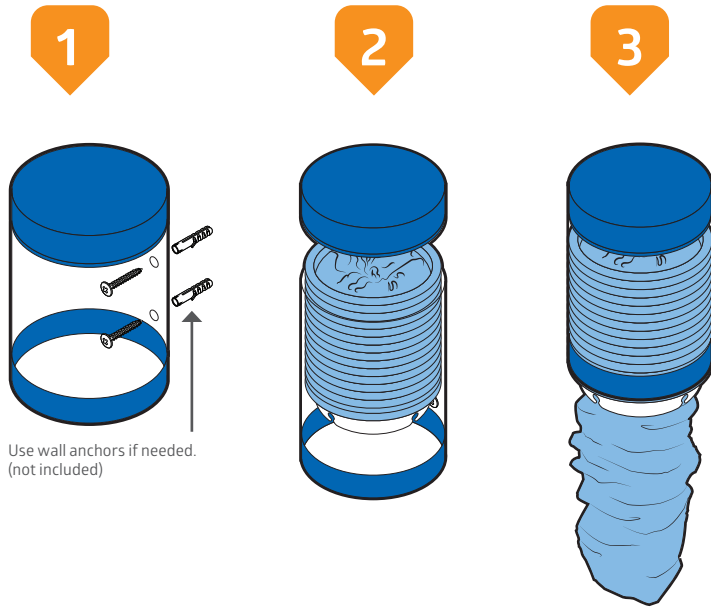

Use wall anchors if needed.
(not included)


- 1. Install**—affix the dispenser in a convenient, easy-access location.
- 2. Fill**—remove top and push one 24-bag pack into the dispenser with the tapered white edge facing down.
- 3. Dispense**—to dispense pull one bag at a time from below, grabbing the center.

Easy to use and secure

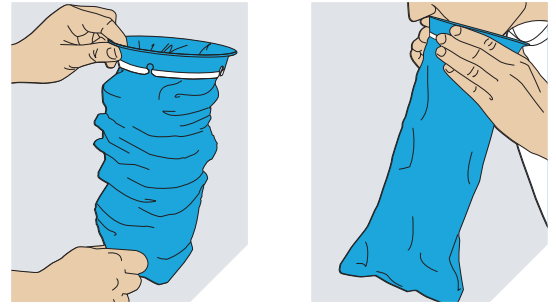

1. Prepare—holding the comfort ring, unfold the plastic bag by pulling it down gently, away from the comfort ring.

2. Use—cover the mouth with the comfort ring during regurgitation, directing the vomitus into the bag until finished.

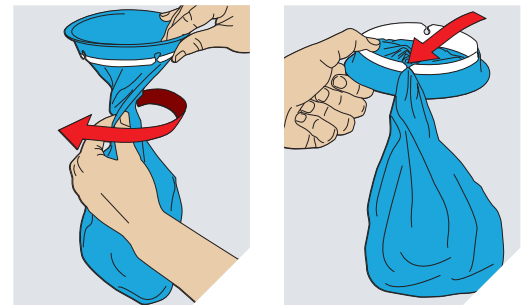

3. Secure—grip the bag 3-4 inches below the comfort ring and twist the ring. Tuck the tightly twisted bag into a notch on the ring to hold. Dispose of in a sanitary manner.

Ordering Information

Item No.	Description	Color	Material	Pkg.
NON80328	Clean Sack Emesis Bag w/ Rigid Ring and Graduations	Blue	LDPE	144/gr
NON80327	Clean Sack Economy Emesis Bag w/ Rigid Ring and Graduations	Green	HDPE	144/gr
NON80329	Clean Sack Emesis Bag w/ Rigid Ring and Graduations	Green	HDPE	144/gr
NON70600	Clean Sack Emesis Bag w/ Paper Funnel and Graduations	Clear	LDPE	500/pk
NONEMBGDISP	Medline Dispenser for Emesis Bags	Blue	—	6/cs
EMEBAG	Centurion Emesis Bag w/ Rigid Ring and Graduations	Blue	HDPE	144/cs
EMEBAGCL	Centurion Emesis Bag w/ Rigid Ring and Graduations	Clear	HDPE	144/cs
EMED200G	Centurion Dispenser for Emesis Bags	Green	—	6/cs

Ask about our free dispenser program. For program and product details, contact your Medline Representative.